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**REF:** BTHM

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**Revisão nº** 00

**Product's name:** Filtering Half Mask FFP2 NR

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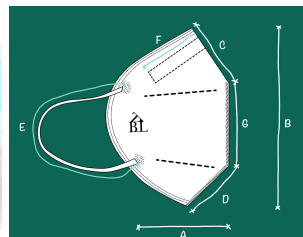
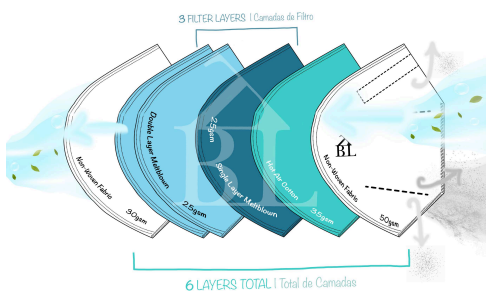
Disposable face mask with a 3D structure, thanks to the vertical pleat, giving it more comfort. With six layers of protection it comes with elastic ears strings. It has a nose clip on the inside to better fit the wearer's face. Not sterile.

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**Raw Material:** White Non-woven fabric 80g (1 TNT layer 50g and another 30g), Meltblown 98-99% BFE 50g (2 layers 25g), Hot Air Cotton 35g (1 layer), Elastic for the ears (5mm), Plastic nasal wire (5mm)

PARAMETERS		SIZES (mm)
A	LENGTH	105
B	TOTAL HEIGHT	160
C	LENGTH TOP	55
D	LENGTH BOTTOM	65
E	ELASTIC LENGTH	170
F	NASAL CLIP LENGTH	95
G	HEIGHT OF PLEAT	70

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**Conservation:** Store masks out of direct sunlight, away from chemicals and in a dry atmosphere.

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**Intended Use:** Alternative masks for frequent contact with the public, for single use

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Before use you should wash your hands and make sure the mask is not damage.

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1. Hold the mask with the nose strip up and the FFP2 mark facing outside.
2. Put the mask over your nose and mouth and make sure you cover them fully.
3. Secure the ear strings and adjust for fit comfortably..
4. Press the nose strip for perfect fitting to your face.
5. Cover the mask with both hands without pinching the shape and inhale and exhale. If you feel air blowing from the edges readjust it.

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Category III PPE manufactured in accordance with Regulation (EU) 2016/425, more specifically with the norm EN 149: 2001 + A1: 2009. Respiratory protection device or Filtering half mask to protect against particles. Requirements, tests, marking, in accordance with Certificate nr. 21/4743/00/0161 by AITEX (NB 0161).

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